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
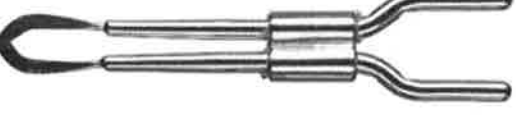
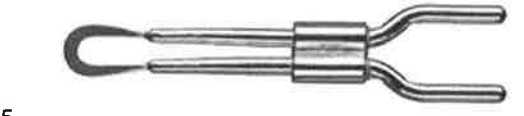
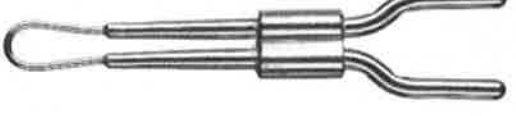
6
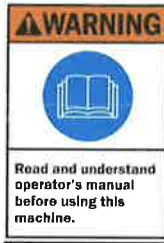
## Table of contents

Intended use.....	
Scope of delivery.....	
Safety information.....	
Commissioning.....	
Technical data.....	
Disposal.....	
Warranty.....	
Declaration of conformity.....	
Replacement cauterizing tips.....	

## Intended use

The **Connector World Trade - Electro Cauterizer** may only be used for cauterizing purposes.

The cauterize tips can be replaced to your preference.

Any other use is not permitted and may result in injuries.

In these cases, there shall be no liability on the part of the manufacturer.

## Scope of delivery

1x Electro Cauterizer with EU-plug

1x green adapter-set with connection sockets for the cauterize elements.

## Safety information - It is essential to read the safety information !

- This device may only be used by qualified professionals.

- Make sure that the power cord does not come into contact with heat, oil or sharp edges. Damaged power cords can cause fires, short circuiting and electric shocks.
- If the power cord of this device is damaged, it must be replaced by the manufacturer to avoid danger.

- Failure to use the device with care may result in a fire. Try to prevent accidents, do not use the device in a potentially explosive atmosphere, remove all flammable materials/liquids before use, use a heat-resistant work surface and keep your working area clean.
- Before use, inspect the device for damage and check that the cauterize element is firmly in place in the correct position. If damaged, do not use; contact customer service.
- For reasons of safety and licensing (CE), the unauthorized modification and/ or alteration of the Electro-Cauterizing Equipment is not permitted.

- Protect the tool from liquids and moisture, as these can cause fires, short circuiting and electric shocks.
- Do not leave the device unsupervised while it is operating. During longer work interruptions, the device should be disconnected from the power supply.
- **Caution:** After use, allow the tool to cool down before putting it away, because there is a fire risk until the hot element of the cauterizer has cooled down.
- When the device is not in use, store it in a safe place and protect it from dust and moisture.
- National and international occupational health and safety provisions must be observed.
- **Always Beware:** there is a danger of fire when working with flammable disinfectants

## Commissioning

1. Release the two clamping screws; insert the green adapter-set and tighten the screws again, now press the cauterize- element firmly into the connection sockets of the green adapter-set.
2. Plug the supply cord with EU-plug into a suitable socket.
3. Press the pistol-trigger; the cauterize-element is heated and illuminated by the lighting.
4. After 6 seconds, the cauterize temperature has been reached.
5. Now it's ready for use, after use, pull the plug out of the socket.

**Technical data - Connector World Trade - Electro Cauterizer**

**Operating voltage:** 230V~ 50-60Hz

**Power consumption:** 80W

**Lighting:** 6Volt - AC (2 units)

**Heat up time:** 6 seconds

**Intermittent operation:** 12 sec. on / 48 sec. off

**Dimensions:** 29x13 x5.4 cm

**Weight:** 1 kg

The device is not intended for continuous use, but for intermittent operation.

The heat up time is 6 seconds.

The operating period (under suitable heat dissipation conditions) is 12 seconds.

The device must then be turned off to cool down for 48 seconds.

**Disposal**

The "crossed-out dustbin" symbol requires separate disposal of electrical and electronic equipment (WEEE). Electrical and electronic equipment may contain dangerous and environmentally hazardous substances. Do not dispose of this device or its packaging with household waste. Dispose of the device in a designated collection point for waste electrical and electronic equipment. By doing this, you are contributing to conserving resources and protecting the environment. Please contact your dealer or local authorities for further information.

**Warranty**

You have 1 year's warranty on **this Connector World Trade - Electro Cauterizer** from the date of purchase.

This warranty covers all defects that result from possible material or production faults.

The warranty shall be invalidated in the case of improper use or misuse.

**Declaration of conformity**

We declare that the product described under the Technical data complies with the following directives:

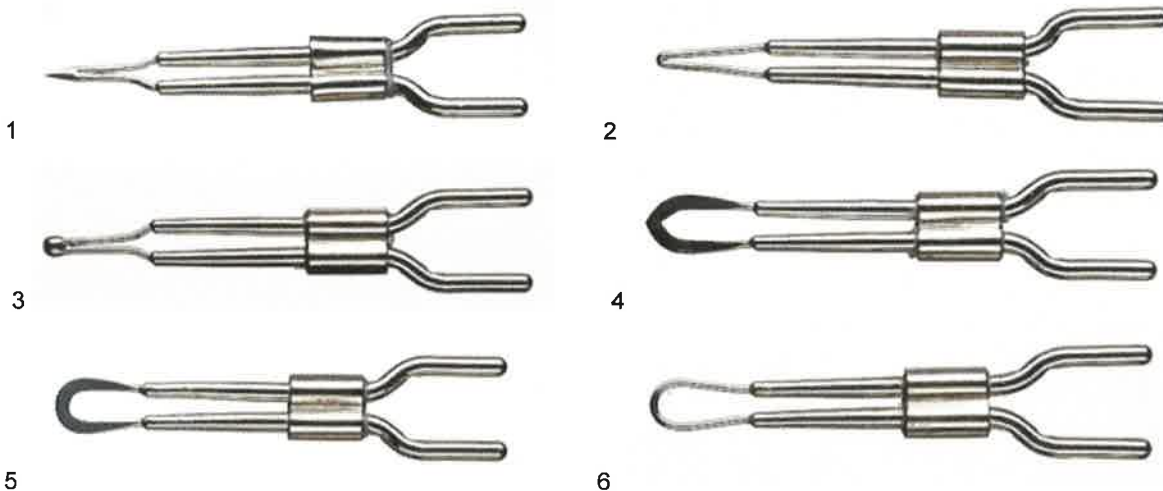
EMC Directive 2014/30/EU

Low Voltage Directive 2014/35/EU

RoHS Directive 2011/65/EC

**Replacement cauterizing tips**

There are 6 different types of cauterize elements:

**CONNECTOR**  
WORLD TRADE

**Connector World Trade**  
The Netherlands  
Europe  
info@cwtnl.com